
How to collect a nasal sample from home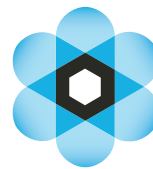

CIRRUSDX

Your home collection kit contains: (1) specimen bag,
(1) test requisition form, (1) sterile swab and NEST® tube labeled
with patient name and DOB, (1) pre-paid FedEx clinical return pak

Follow these steps:

Check	Make sure your information is correct on the test requisition form and Nest® tube.
Write	Write the date you collect your sample in the highlighted area on the test requisition form.
Clean	Thoroughly wash your hands.
Remove	Carefully remove the swab from the packaging without touching the soft tip.
Collect	(1) Insert the tip of the swab into one nostril until the tip of the swab is no longer visible. (2) Rotate in a circular motion around the entire inside of the nostril for at least 10 seconds. (3) Take the swab out of the nostril and repeat for the other side. (4) Break swab at the scoreline, place it into the supplied NEST® tube, and screw cap on firmly.
Pack	Place your sample in the specimen bag and seal it. Put your folded requisition form and sealed specimen bag inside of the pre-paid FedEx clinical return pak.
Send	Call 1-800-GO-FEDEX to arrange pick up of your sample. Or drop it off in a FedEx box or at a FedEx shipping center.*

*Please note, samples shipped out on
Fridays will be accepted for Saturday
delivery and testing.

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