
# How to collect a vaginal sample from home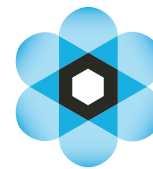

**CIRRUSDX**

Your home collection kit contains: (1) specimen bag, (1) test requisition form, (1) sterile swab and NEST<sup>®</sup> tube labeled with patient name and DOB, (1) prepaid clinical return pak

## Follow these steps:

<b>Check</b>	Make sure your information is correct on the test requisition form and Nest <sup>®</sup> tube.
<b>Write</b>	Write the date you collect your sample in the highlighted area on the test requisition form.
<b>Clean</b>	Thoroughly wash your hands.
<b>Remove</b>	Carefully remove the swab from the packaging without touching the soft tip.
<b>Collect</b>	(1) Carefully insert swab into vagina approximately 5 cm (2 in) past vaginal opening. (2) Gently rotate swab against the vaginal wall for about 10–30 seconds. (3) Break swab at the scoreline, place it into the supplied NEST <sup>®</sup> tube, and screw cap on firmly.
<b>Pack</b>	Place your sample in the specimen bag and seal it. Put your folded requisition form and sealed specimen bag inside of the prepaid clinical return pak.
<b>Send</b>	Call <b>1-800-GO-FEDEX</b> to arrange pick up of your sample. Or drop it off at a FedEx Office shipping center.* Call <b>1-800-PICK-UPS</b> to arrange pick up of your sample. Or drop it off at a UPS Store location.*

\*Please note, samples shipped out on Fridays will be accepted for Saturday delivery and testing.

### Questions? We're here to help.

Call **1-240-813-8801** Monday through Friday, 8 AM to 9 PM ET.

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©2026 CirrusDX, Inc. Todos los derechos reservados